



PerioRenew

Bio Oxidation Tray Making Instructions and Usage

Tray Fabrication

- Use .080 EVA or thicker/stiffer tray material
- On model draw a line 2-3 mm from the gingival margins.
- Use a #1 burr to create a shallow trench on the model.
- When vacuumformed this creates a seal/dam on the tray. This promotes subgingival hydraulic pressure with the gel.
- Trim trays 4-5 mm beyond the gingival margin, onto the soft tissue.
- When the patient places PerioRenew™ gel in their personal oxidation tray, then pushes the tray into place over the teeth, they are creating hydraulic pressure that forces the gel under the gum line where tooth brushes and mouth wash do not reach. Biofilm bacterial activity is then interrupted.



Regimen

12 weeks; recall in 90 days.

Dosage

Squeeze a thin line of PerioRenew™ (2% hydrogen peroxide) into the tray.

Treatment Time

Apply for 5 to 15 minutes per application, daily (as your doctor recommends).

Useful Information

- This will help people with dexterity problems who have a hard time brushing and flossing hard to reach places.
- An effective cleaning therapy before and after dental procedures.



Life-Like Cosmetic Solutions, Inc.

51 Santa Felicia
Goleta, CA 93117

1-800-543-3545
1-805-962-0333
www.life-like.com